



# De Mayo Hip Positioner® System STANDARD Set Up Instructions

1. Leave OR table pads in place, then place OR bedsheet on top.




2. Pre-Assemble the Positioner. Assemble flat bar into bottom groove of Lumbar Support Base. Assemble washer and locking knob.




3. Roll the patient on their side.




Attach Contoured Lumbar Pad for left or right hip application.

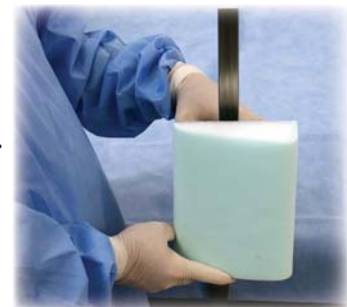

4. Assemble Lumbar Support Base into the imp® Rail Clamp at a position even with the patient's iliac crest. Use the Vertical Bar in a straight and upright position as a guide to aim at the patient's iliac crest.

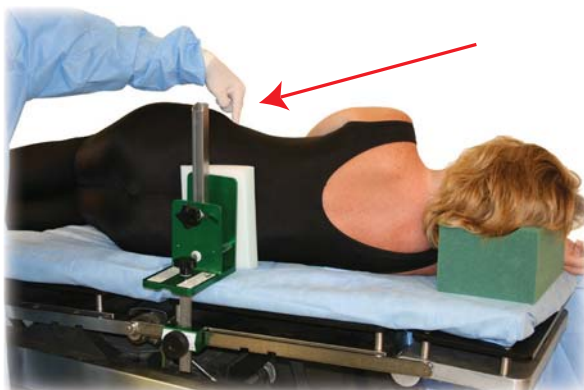



Lock Flat Bar Knob in center groove of Support Base.

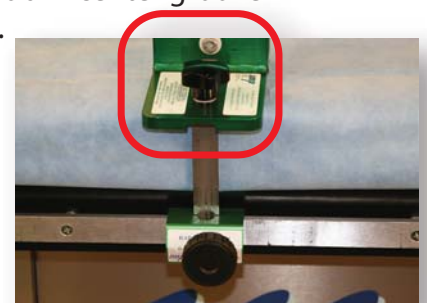

# De Mayo Hip Positioner® System

## STANDARD Set Up Instructions page 2

5. Angle the Vertical Support Bar towards the head of the patient. Assemble the Horizontal Crossover Standard Bar #1 onto the Vertical Bar #1. Allow 2 fingerbreadths space between the patient and the Bar.

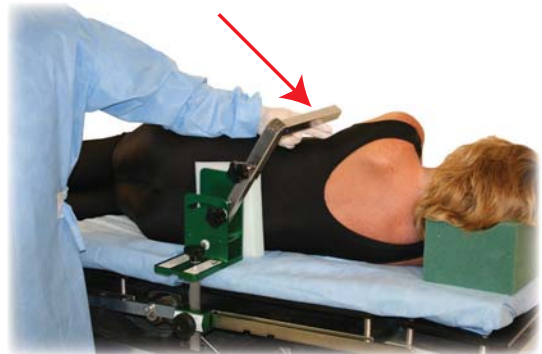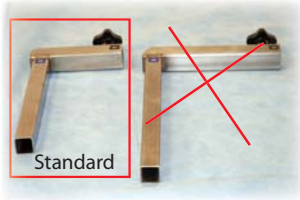


6. Place Anterior Patient Protector Pad.




Attach pre-assembled #2, #3, #4 Anterior Support Assembly\* on Crossover Bar #2.



\*Anterior Support Assembly (includes Anterior Support, Anterior Vertical Support (Std), Anterior Pelvic Support)

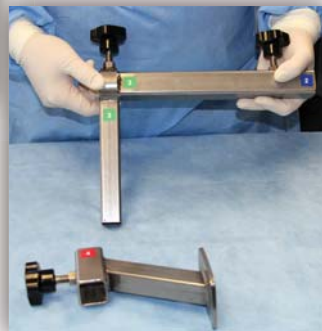

CAUTION: Make sure the threaded knob fits into groove on #3

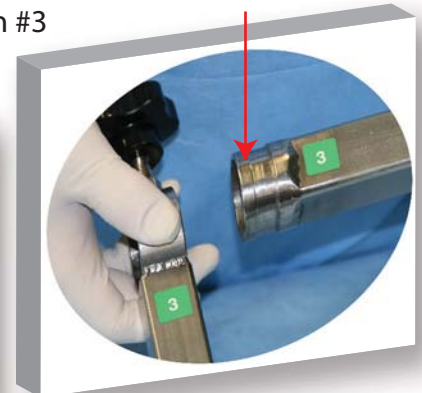


# De Mayo Hip Positioner® System STANDARD Set Up Instructions page 3

7. Rotate Anterior Vertical Bar #3 and adjust Anterior Pelvic Support #4, so that the Anterior Patient Protector Pad contacts the superior iliac crest.



8. Compress Anterior Support Assembly pulling toward Lumbar Support. Lock Anterior Support Knob #2.

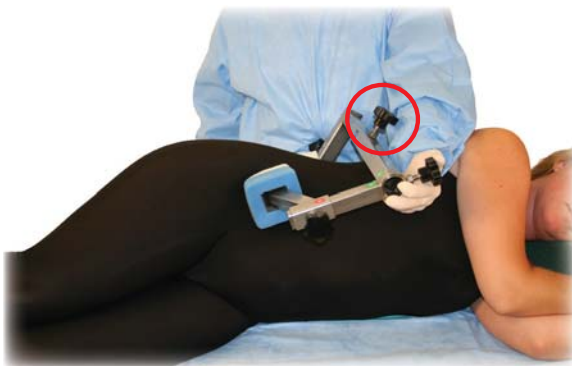

TIGHTEN ALL KNOBS FOR SECURITY



If any readjustments are required,  
release Compression Knob #2, then adjust  
any other knobs to achieve greatest stability.

# De Mayo Hip Positioner® System

## STANDARD Set Up Instructions page 4

On very small patients, the Adjustable Lateral Support (707) with Patient Protector Pad (707-P) can be used to contact the patient's inferior iliac crest to prevent pelvic tilt and movement.



Slide the Vertical Bar into the imp® Rail Clamp.



Adjust Horizontal Bar to make contact with the patient's inferior iliac crest.



*imp*® **Innovative**  
Medical Products, Inc.

*A GLOBAL LEADER IN PATIENT POSITIONING*

*87 Spring Lane, Plainville, CT 06062*

*PH: 860-793-0391 FAX: 860-793-8975*

*Toll Free PH: 800-467-4944 FAX: 888-229-1452*

*www.impmedical.com*