

# *imp* UNIVERSAL LATERAL POSITIONER™

## *SETUP INSTRUCTIONS*

1. Attach Patient Pads to the Universal Lateral Positioner and place Positioner on O.R. table.
2. Transfer patient onto the base of the Positioner and place in the lateral position, aligning patient's iliac crests with the center groove on the Positioner base plate.
3. Place T-Lock over the posts of the Positioner and secure to the side rail of the O.R. table with the Side Rail Clamp.
4. With the patient in the lateral position, slide the Lumbar Support into the groove and advance the support to the edge of the Positioner base plate.
5. Slide the Anterior Support Unit into the proximal groove of the Hyperflexion Plate to the end; back off 1" to 1-1/2" and tighten knob. Then slide the Hyperflexion Plate and Anterior Support assembly onto the groove of the Positioner base plate toward the iliac crests.
6. Adjust the Superior Anterior Arm and Pad to the appropriate height for contact with the superior iliac crest. Then adjust the Inferior Arm and Pad to contact the inferior iliac crest. (If needed, remove the inferior arm to accommodate the anatomy of the larger patient.)
7. Compress the Anterior Support to stabilize the patient in the lateral decubitus position.
8. Final assessment of pad contact and vertical position of patient is achieved by readjusting the Anterior and/or Lumbar Supports to the desired position.
9. Proceed to drape the patient in the standard fashion.



*A GLOBAL LEADER IN PATIENT POSITIONING*

*87 Spring Lane, Plainville, CT 06062*

*PH: 860-793-0391 FAX: 860-793-8975*

*Toll Free: PH: 800-467-4944 FAX: 888-229-1452*

*www.impmedical.com*