

De Mayo Universal Comfort Finger Trap™ Instructions For Use ~ Sterile Technique



1. Remove sterile Trap from packaging.



2. Completely unwrap Velstretch® strap.



3. Seat Trap completely over desired finger.



Hook-tab must be exposed, not between fingers.

Pull strap entirely around the first finger, attaching Velstretch® strap to hook material at the base of the finger.





4. Wrap strap around adjacent finger and attach Velstretch® to top hook material.



5. Apply second Trap to another finger using the same technique.



Loops



6. Suspend arm by utilizing the attached "Loops"

~ OR


for further adjustment utilize the attached "Cords" in "V" Groove.



Cords



Set Up Test Procedure

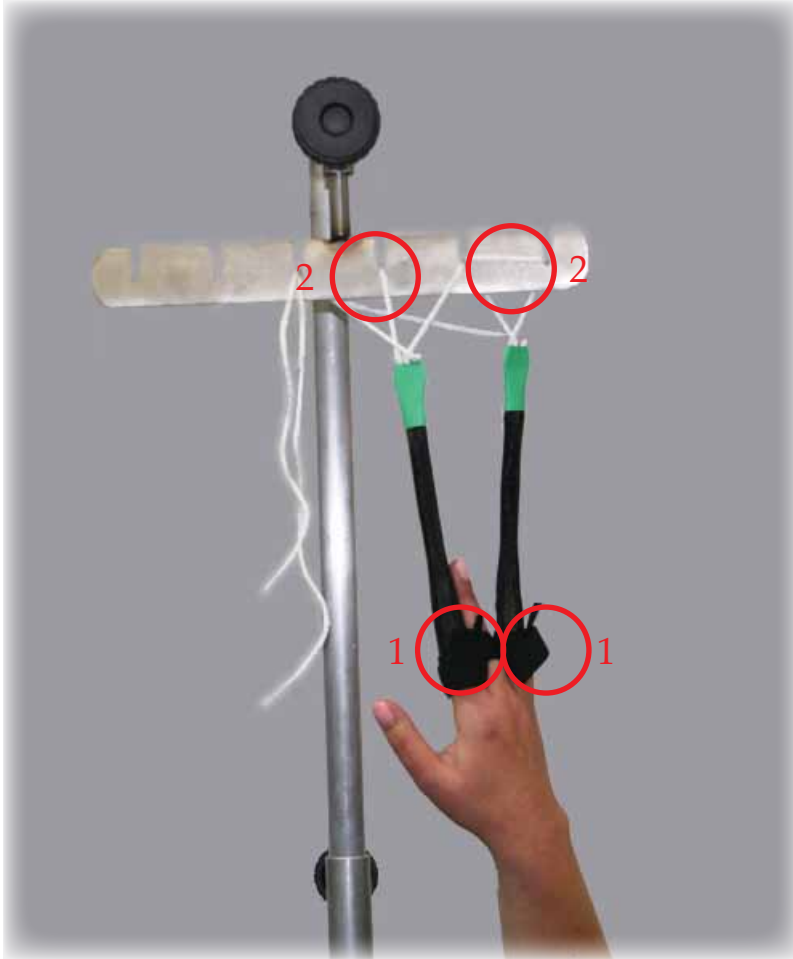

1. Check that Velstretch® Strap is securely attached to Hook material.

2. Check that Loop/Cord is securely attached to Finger Trap holder.



STERILE EO



imp Innovative
Medical Products, Inc.

87 Spring Lane, PO Box 8028
Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944

www.impmedical.com

De Mayo Universal Comfort Finger Trap™ is a trademark of Innovative Medical Products, Inc.
U.S. Patent No. 8,839,797

Velstretch® is a registered trademark of Velcro Industries B.V.

© July 2015 Innovative Medical Products, Inc.

ALL RIGHTS RESERVED