

imp® Humbles LapWrap® Positioning Pad Set Up Instructions



1. Place LapWrap® on OR table pads, VELCRO® brand *black* hook fastener side UP.


2. Place patient so the upper edge of the LapWrap® is located in the axillary region.



3. Wrap entire 3 piece strap around the patient's arm and attach VELCRO® brand hook-and-loop fastener near or under patient.



4. For unusual table positions, patient can be secured to the table by placing center tongue between OR table and side rail. Wrap tongue around side rail and attach VELCRO® brand *black* hook strip to VELCRO® brand *green* loop.

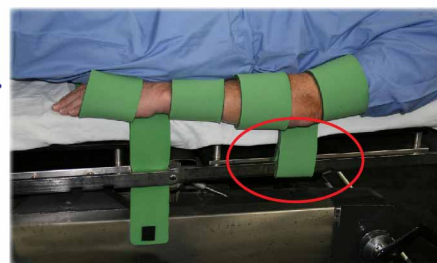

FOR BARIATRIC PATIENTS:

1. Lift LapWrap® center tongue of the upper arm and pull between the torso and the biceps.



2. Insert the tongue out through the cutout, while pulling the VELCRO® brand hook-and-loop fastener around the outer arm and towards the body.



3. Fasten to the center tongue.



Optional Technique: Place TUFFease® pressure redistribution pad on OR table for extra patient protection.



Universal Design Fits All Size Patients

Always use appropriate positioning safety precautions



Our product features genuine VELCRO® brand fasteners.

LapWrap® is a registered trademark of Innovative Medical Products, Inc.

LapWrap® - U.S. PATENT No. 8,001,635

TUFFease® is a registered trademark of Innovative Medical Products, Inc.

VELCRO® is a registered trademark of Velcro Industries B.V.

© 2012 Innovative Medical Products, Inc. ALL RIGHTS RESERVED

REV 01/2012

imp® Innovative

Medical Products, Inc.

87 Spring Lane, PO Box 8028

Plainville, CT 06062

PH: (860) 793-0391 FAX: (866) 459-1805

Toll Free PH: 1-800-467-4944 FAX: 1-888-229-1452

www.impmedical.com