

imp MORPHBOARD® POSITIONING SYSTEM

STANDARD APPROACH TO LATERAL DECUBITUS POSITIONING SET UP INSTRUCTIONS

1. Remove OR Table Pads




(Please note it is not necessary to remove the X-ray cassette plates)

2. Place the 3 Boards onto the surgical table




making sure to interlock the 2 End Boards with the Center Board

3. Slide the 2 side rail clamps onto the surgical table rail, and position each of them where the boards interlock

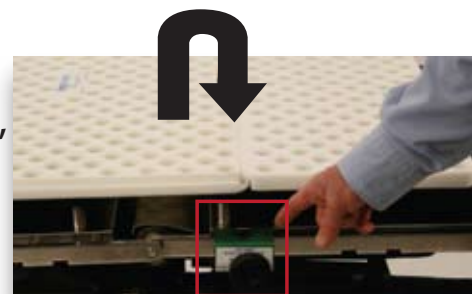


4. Slide the vertical bars of the clamping plates into the side rail clamps, bridging the interlocking grooves of the End Boards with the Center Board.

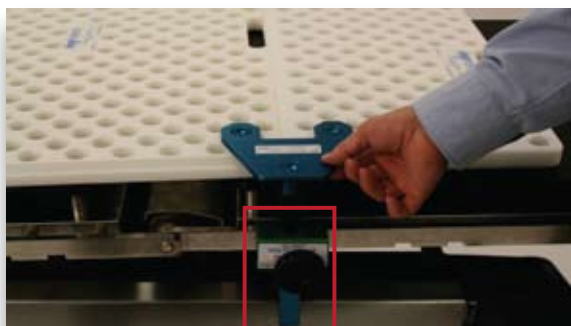

It is important that the clamping plates "bridge" the interlocking grooves.



5. Tighten the side rails clamps to lock each clamping plate

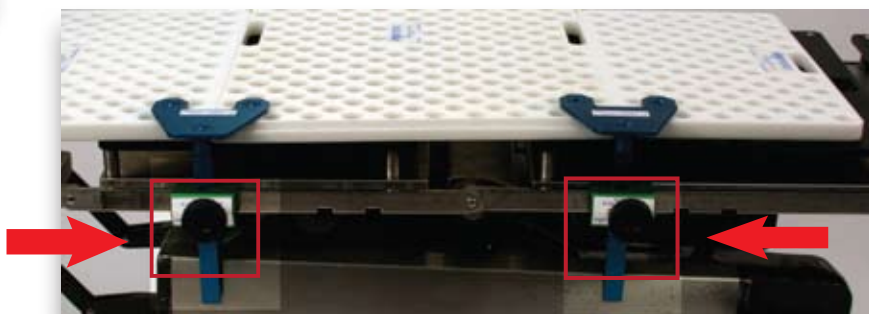

imp MORPHBOARD® POSITIONING SYSTEM

STANDARD APPROACH TO LATERAL DECUBITUS POSITIONING

SET UP INSTRUCTIONS CONTINUED

6. Place the patient protective pad onto the boards.

Innovative Medical Products provides 3 options:



TUFFease® Gel Pad



Slow Recovery Foam




Combination of a Foam Pad
onto a Gel Pad



You may now place a draw sheet on top of the protective pads to assist with positioning the patient.


7. Place the patient in the lateral position.

Place foam covers on pegs and use in the following locations:

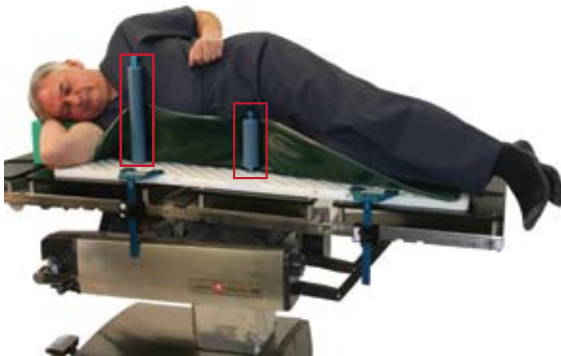

Anterior pegs should be placed with one 8" peg in front of the pubis. Care should be taken to avoid compression of the neurovascular structures. A short peg is used to maximize the range of motion. Next, insert one 14" peg directly superior to the pectoral muscle.

Posterior pegs should be placed with one 12" peg in the lumbar region, and one 14" peg in the thoracic region below the scapula

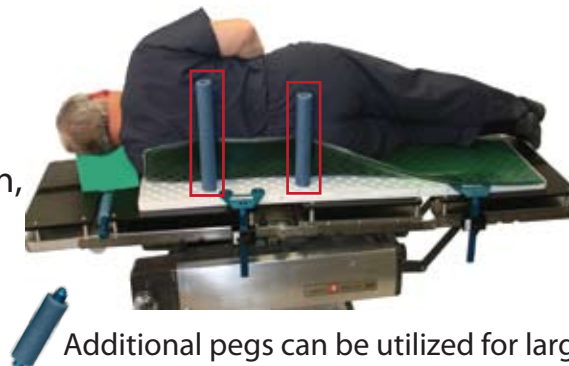

Additional pegs can be utilized for larger patients

Set Up is now complete for hospital standard draping and prepping

Cleaning of boards & pegs can be accomplished with a standard hospital antiseptic solution

imp Innovative

Medical Products, Inc.
87 Spring Lane, PO Box 8028
Plainville, CT 06062
PH: (860) 793-0391 FAX: (860) 793-8975
Toll Free PH: 1-800-467-4944 FAX: 1-888-229-1452
www.impmedical.com



MorphBoard® is a Registered Trademark of Innovative Medical Products, Inc.
TUFFease® is a Registered Trademark of Innovative Medical Products, Inc.
© 2009 Innovative Medical Products, Inc.
ALL RIGHTS RESERVED